

CUBICLE TRACKING ASSEMBLY INSTRUCTIONS

6000 SERIES

Ceiling Mounted Track

Cut track to length and attach end stops (use End Stops in pairs: one with a pull-out, and one without). Secure track to ceiling. Once track is securely mounted, remove pull-out from end stop, insert the required number of carriers, and re-insert pull-out into the end stop. To install the cubicle curtain, carefully slide the grommets, located at the top of the cubicle curtain, over the hooks of the carriers.


For smoother operation, lightly spray the inside of the track with an aerosol silicone lubricant. Depending upon environment, clean and re-lubricate as necessary.

Suspension Mounted Track

Cut track to length and attach end stops (use End Stops in pairs: one with a pull-out, and one without). If either of the track ends will be butting directly into a wall, replace the necessary End Stop(s) with a Track End Wall Bracket. For details, refer to Track End Wall Bracket assembly instructions. Attach suspension tubing ceiling brackets to the ceiling then attach suspension tubing and track brackets. Position suspension tubing assemblies along path of the track at 24" to 36" intervals. Once the suspension tubing is securely mounted to the ceiling, attach the track to the suspension tubing. Once the track is secured, remove pull-out from end stop, insert the required number of carriers, and re-insert pull-out into the end stop. To install the cubicle curtain, carefully slide the grommets, located at the top of the cubicle curtain, over the hooks of the carriers.

For smoother operation, lightly spray the inside of the track with an aerosol silicone lubricant. Depending upon environment, clean and re-lubricate as necessary.



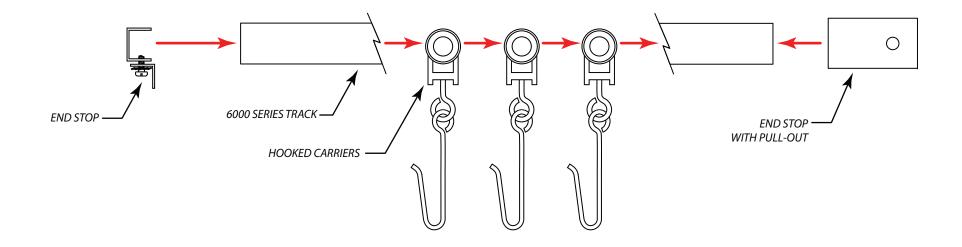

CONFIDENTIAL

All materials disclosed in this document are proprietary to Diamond Drapery Co, Inc. The Holder agrees to maintain it in confidence and must not replicate or reveal this information in any manner.

Project:

6000 Series Tracking System Assembly


DO NOT SCALE

10 010 2000	micries, reet
10-DLC-2006	iliciles/reet
16-DEC-2008	Inches/Feet
Date:	Dimensions:

C. Johnson

DIAMOND

7439 Lampson Avenue Garden Grove, CA 92841 (714) 761-4543 OFFICE (714) 761-4623 FAX

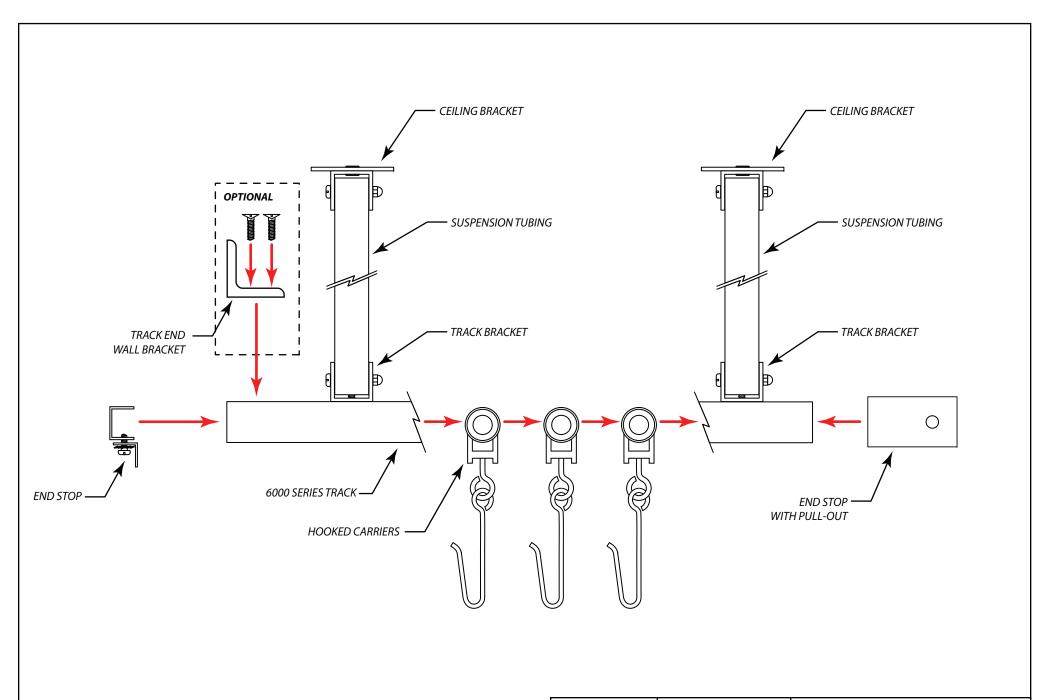

CONFIDENTIAL

All materials disclosed in this document are proprietary to Diamond Drapery Co., Inc. The Holder agrees to maintain it in confidence and must not replicate or reveal this information in any manner.

Project:

6000 Series Suspended Tracking System Assembly

Date:	Dimensions:
16-DEC-2008	Inches/Feet
Scale:	Rv

C. Johnson

DO NOT SCALE



7439 Lampson Avenue Garden Grove, CA 92841 (714) 761-4543 OFFICE (714) 761-4623 FAX



CUBICLE TRACKING ASSEMBLY INSTRUCTIONS


6000 SERIES CUBICLE TRACK - TRACK END WALL BRACKET

A wall bracket (part #6000-07) is used to attach the ends of suspension mounted tracks to the wall and acts as a closed end stop. This mounting greatly enhances the integrity of a suspension mounted cubicle tracking system.



Instructions:

 Once the suspended track system is in place, locate the track ends and mark their position on the wall. If an end stop is already installed, remove it, attach the Wall Bracket using the supplied screw and reinstall the end stop. The end stop may need to be crimped a small amount with a pair of pliers so that it will clear the Wall Bracket screw.

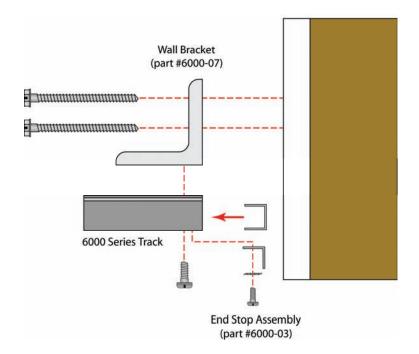

2. Attach the Wall Bracket to the wall using the appropriate mounting method (drywall & stud method illustrated here).

